

VALIDOSS


SAMPLING KIT

FOR OLYMPUS®

These instructions for use are a shortened version of the complete manual which can be found at validoss.nl/Sampling-Kit-IFU/

This sheet serves only as a reminder.

INSTRUCTIONS FOR USE

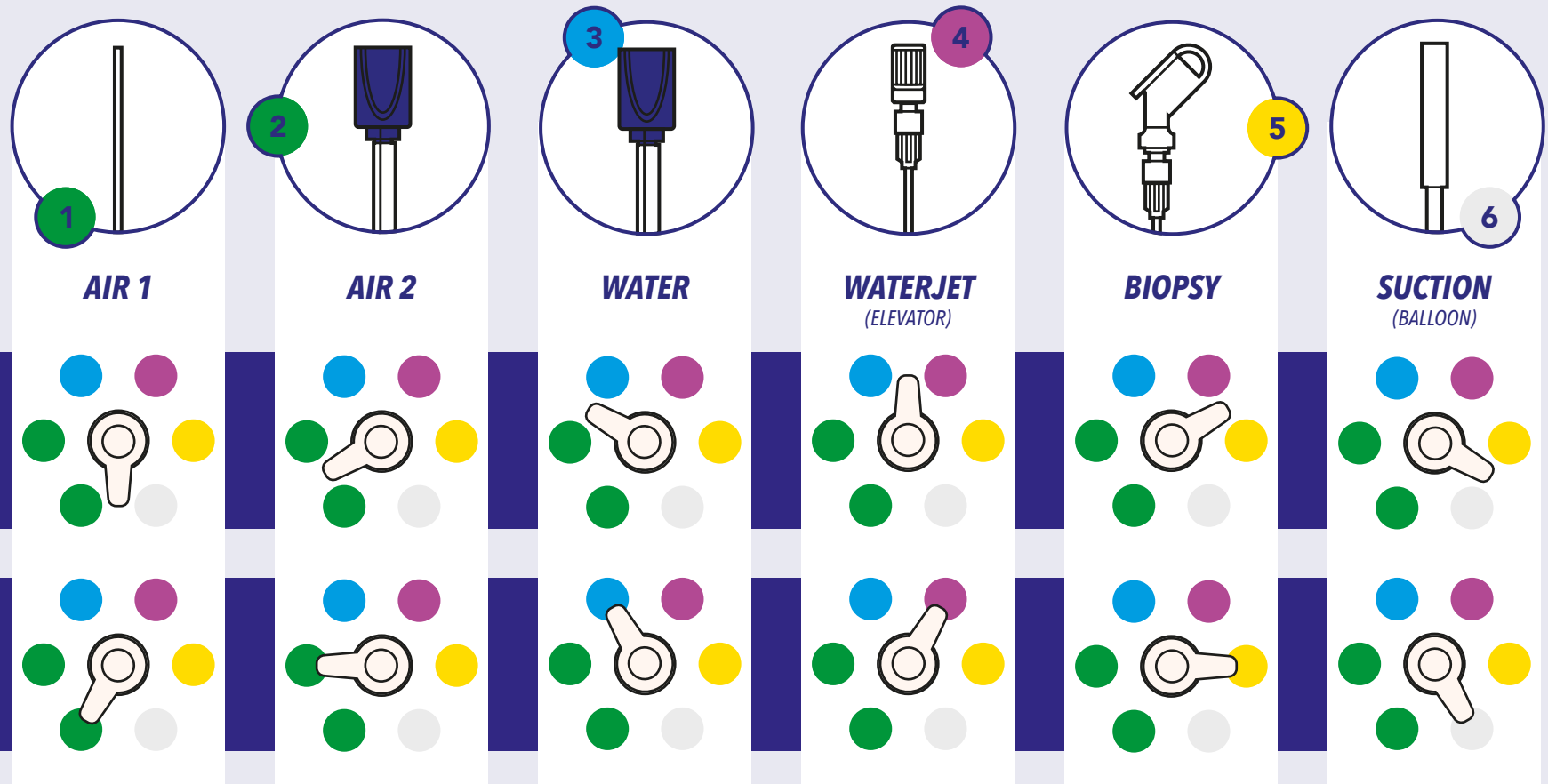


ATTENTION


- Ensure that excess rinsing fluid is lightly tapped off from the tip of the endoscope between each sample collection.
- Always hold the sample jars beneath the tip of the endoscope. Be mindful not to let the tip touch the inside of the sample jar.
- Do not apply too much pressure on the channels during flushing, especially with the air and water channels.
- Ensure registration and documentation for each sample jar, collecting a minimum of 20mL.

PREPARATION


Prepare a clean workspace and work aseptically. Place the endoscope to be cultured in a transport container and have the following tools ready:

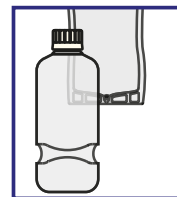

rinsing fluid
(bottle or bag)

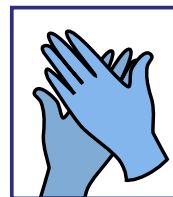

(sterile) gloves

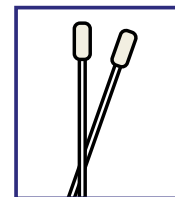

swabs



sample jars

SUPPLY AIR

Turn the blue valve as shown below to fill the syringe with air and then push it through the channel.

