

# VALIDOSS


## SAMPLING KIT

FOR PENTAX®

These instructions for use are a shortened version of the complete manual which can be found at [validoss.nl/Sampling-Kit-IFU/](http://validoss.nl/Sampling-Kit-IFU/)

This sheet serves only as a reminder.

## INSTRUCTIONS FOR USE

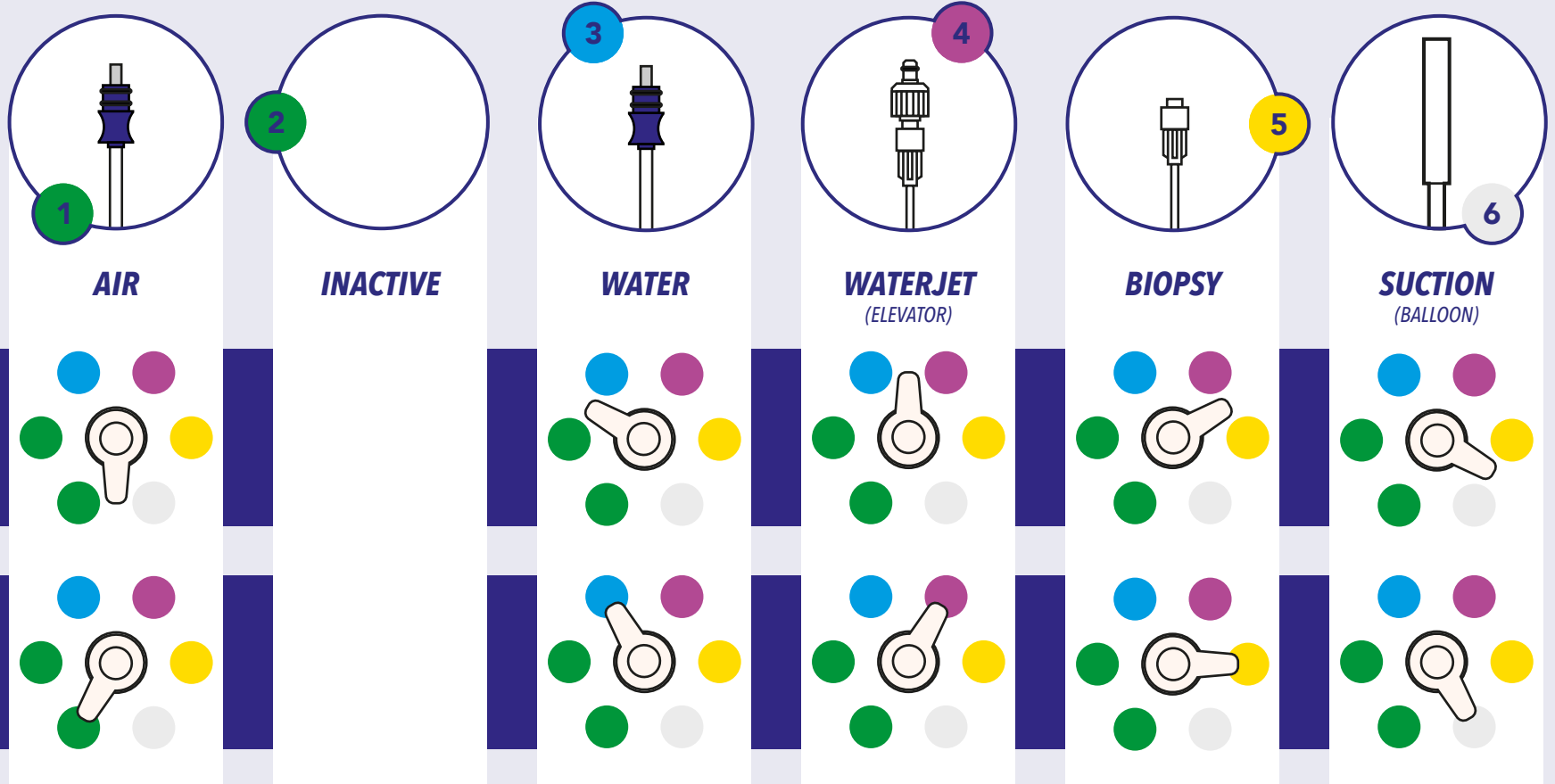


### ATTENTION


- Ensure that excess rinsing fluid is lightly tapped off from the tip of the endoscope between each sample collection.
- Always hold the sample jars beneath the tip of the endoscope. Be mindful not to let the tip touch the inside of the sample jar.
- Do not apply too much pressure on the channels during flushing, especially with the air and water channels.
- Ensure registration and documentation for each sample jar, collecting a minimum of 20mL.

### PREPARATION


Prepare a clean workspace and work aseptically. Place the endoscope to be cultured in a transport container and have the following tools ready:

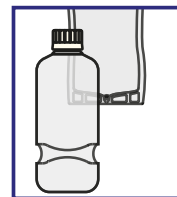

rinsing fluid  
(bottle or bag)

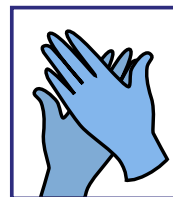

(sterile) gloves

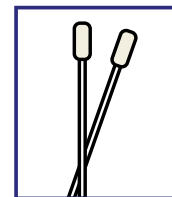

swabs



sample jars

### SUPPLY AIR

Turn the blue valve as shown below to fill the syringe with air and then push it through the channel.

